Main features

Breast milk is the best choice for baby, which contains many antibodies and is the source of baby's immunizator. But sometimes mommy will find it is inconvenient to feed baby directly. Our breast pump is mommy's ideal appliance which can imitate rhythm of baby's sucking, and its soft silicone cushion can breast a massage, alleviate pain and avoid the nipple collapses. Our breast pump makes nursing mommy healthier and more beautiful.

Massage and sucking 2 in 1 function
Easy massage and strong suction power, two adjustable mode for efficient breast milk collection


Adjustable large suction power
Large suction power for smooth expressing, adjustable suction power for different options.

Safety material
Food grade PP and silicone material, BPA free

Easy to assemble
Cute design and convenient to carry out.

List of components

No.	Name	Materia	Heat	Cold	Quantity
A	soft breast shield	Silicone Rubber	200 C	-20 C	1
B	Funnel unit	PP	120 C	-20 C	1
C	Suction knob	POM	110 C	-20 C	1
D	Diaphragm	Silicone Rubber	200 C	-20 C	1
E	Stem	POM	110 C	-20 C	1
F	Dustproof cover	PP	120 C	-20 C	1
G	Handel knob	PP	120 C	-20 C	1
H	Handle	PP	100 C	-20 C	1
I	Adaptor A	PP	120 C	-20 C	1
J	Milk bottle(150ml)	PP	120 C	-20 C	1
K	Valve Membrane	Silicone Rubber	200 C	-20 C	1
L	Bottle stand	PP	120 C	-20 C	1
	Instruction Manual				1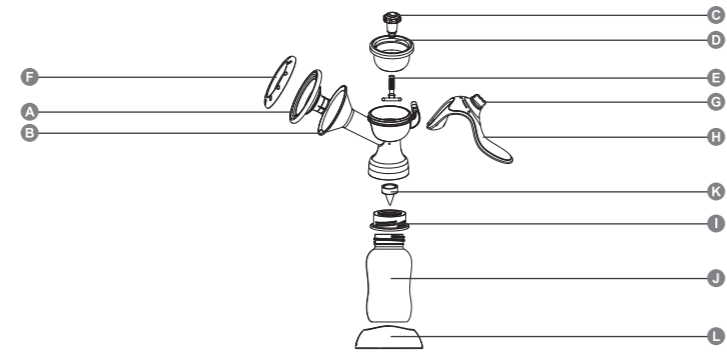


Warning

- Please use this unit under doctor or nursing instruction when mommy in the period of absence of or faulty secretion of milk
- Please make sure to follow the instruction to remove and assembly. Be careful to avoid losing small parts when disassemble or wash.
- This is not a toy, please keep out of reach of children.
- Please adjust the suction power as per individual need.
- Please do not overuse pump in case of breast pain.
- Please consult doctor once feel pain in breast after using.
- Breast pump is just for releasing the pain of breast milk engorgement. Please do not use it in the initiation time of breastfeeding.
- Ensure the parts are assembled correctly.
- Before using, please clean and sterilize parts and keep them clean and dry after use.
- This unit only for the usage stated in this instruction manual.
- It's normal phenomenon to smell plastic. Please clean it correctly and then can decrease the plastic smell.

Cleaning tips


- All components can be sterilized.
- Please clean silicone parts (breast shield, diaphragm, valve and teat) with soft sponge. Faulty cleaning may cause damage and scratch.

Assembly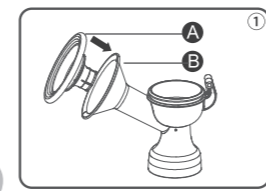


1 Insert the silicone breast shield (A) by aligning the contours of shield to that of the funnel unit (B).


Assembly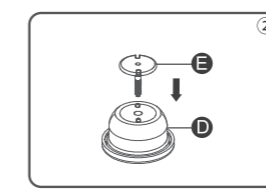


2 Please insert the stem (E) into the silicone diaphragm(D).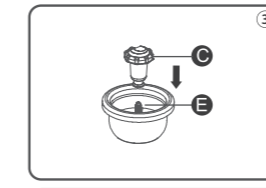

3 Fix the suction knob (C) on stem (E).

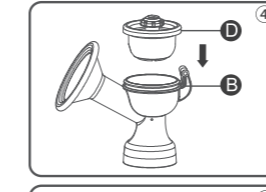
4 Insert the silicone diaphragm (D) into the funnel unit (B) as shown. Ensure these two parts are filled together.

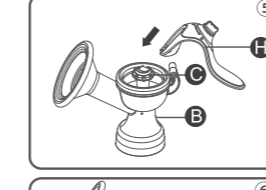5 Slide the stem (C) into the fork of handle (H), and then assemble the handle on the pivot of funnel unit (B).


6 Push the valve membrane (K) into the pump unit (B)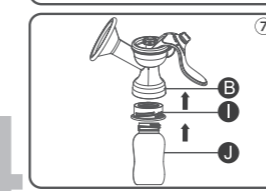

7 Screw the bottle (J) with adaptor A (I) and assembled to funnel unit (B)

Assembly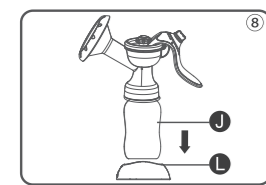

8 Place the assembled bottle (J) onto the bottle stand (L)


*Ensure parts are assembled correctly.

Product Specification

Name	Manual breast pump
Model No	HL-0680

Operating Instruction


- The user should keep comfortable sitting posture and feel relief. It is better to put chair w/footstool under foot or sitting upright in bed.
- Put the assembled breast shield aimed at the nipple and make the breast completely fill shield. (Image.12)
- Open the dustproof cover, please use the adaptor (I) or Funnel unit (B) to match wide neck or standard neck milk bottle.
- Turn the handle knob to massage mode of "☺", or sucking mode "☹". (Image 13)
- As per your feeling, turn the adjustable knob in clockwise direction for strong suction power, otherwise for weak power. (Image 14)
- Use proper strength for handle operation.
- After use, please disassemble the container and seal with its cover or cap. It's better to place the bottle on bottle base in case of overturning. (Image 15)
- Please disassemble and clean some parts after use.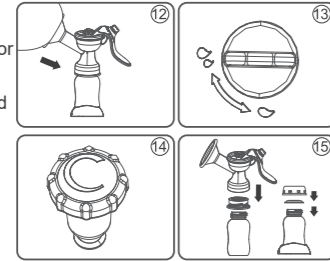

Troubleshooting

Insufficient Suction Pressure

- Ensure nipple is properly filled into breast shield. (Image 16)
- Check the breast shield, diaphragm, stem, handle, valve, etc are assembled correctly. (image 17)

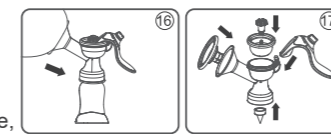How to store and warm breast milk

	Room temperature	Refrigerator	Freezer	Independent freezer
Freshly expressed into closed container	No	3-5 days (4 C or lower)	6 months (-6 C)	12 months (-18 C)
Previously frozen thawed in	No	10 hours	No refrigerator but not warmed or used	

- Only fill container 3/4 full before freezing to allow for milk expansion.
- Add label with time of sucking milk on storage cup or storage bag.
- In case of loosing nutrition, please take out the breast milk stored in freezer and place in refrigerator to thaw, or place in moving warm water (lower than 37 C) to thaw.
- Before feeding, please shake up the breast milk storage cup to mix milk completely.
- Do not thaw breast milk in microwave oven or hot water.

Operation Instruction

Adjustable Manual Breast Pump

Retain this instruction manual for future reference. Read these instructions carefully before usage.

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