SIZE:15.5x24CM

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(GB) USER INSTRUCTIONS KINESIOLOGY TAPE 5mx5cm.

- Decide which muscles you wish to treat.

- Ensure that the skin is clean, dry and grease free.

- Remove excess hair.

- Cut the tape to the required length.

- Remove the protective foil up to the centre of the tape, stretch the tape gently and place it on the respective muscle.

- To prevent blisters: gently stretch the tape and place it firmly.

- Then remove the protective foil from the remaining part, stretch the tape gently and place it on the respective muscle.

- Always rub the tape from the centre to the ends.

WARNINGS:

Do not use with other treatment methods.
(e.g. massage treatment or electrostimulation). In the event of severe injury or complex illness, leave the taping to a physician or physiotherapist.

- Apply the tape thirty minutes before the planned activity.

- Always apply the ends of the tape without stretching it.

- Ensure that the skin is free of dirt, grease and cream.

- Rub the tape firmly on the skin following application.

- Remove any hair growth for an optimal bond.

- Never simply remove the tape from the skin! Soak it carefully with lukewarm water.

- Cut off corners and edges that do not fit.

Maximum wearing time: 2-3 days.

Material: 95% cotton, 5% elastane (latex-free).

- Remove the tape in the case of allergic reactions, red spots or itches and seek medical attention.

- Do not use the tape in the case of plaster allergy, skin illness or thrombosis.

- Always seek medical attention in the case of serious injury.

- Do not apply yourself in the case of children.

Leave the taping to a physician or physiotherapist.

- Keep out of the reach of children.

- Seek medical attention before use if pregnant.

- In order to prevent pinching, do not overstretch the tape.

- Do not use for longer than 3 days.

20°C